

H. Pylori

One Step H. Pylori Feces Test

Wondfo One Step H. Pylori Feces Test is intended for use by healthcare professionals and as qualitative screening *in vitro* diagnostic test for detection of H.Pylori antigen in feces, as an aid in the diagnosis of H.Pylori infection.

99.20%
Sensitivity

99.26%
Specificity



- **Specimen: Feces**
- **Detection Target: H. Pylori anitigen**
- **Detection Time: 15 mins**

CE

Guangzhou Wondfo Biotech Co., Ltd.

No.8 Lizhishan Road, Science City, Luogang District, 510663, Guangzhou, P.R.China

Tel: (+86)400-830-8768

E-mail: sales@wondfo.com.cn

Website: en.wondfo.com.cn

Rev. A1 Rel.:2022/08/18



01. Test Procedure

1 For Test Cassette

2~3 drops

For Test Strip

2~3 drops

2

15 mins

3 For Test Cassette

Control line → C
Test line → T

Positive Negative Invalid Invalid

For Test Strip

C T C T C T C T

Max Max Max

Positive Negative Invalid

02. PERFORMANCE CHARACTERISTICS

Wondfo kit	Commercial kit		Total
	Positive	Negative	
Positive	373	5	378
Negative	3	669	672
Total	376	674	1050

Sensitivity of Wondfo One Step H. Pylori Feces Test: 99.20%

Specificity of Wondfo One Step H. Pylori Feces Test: 99.26%

03. ORDER INFORMATION

Description	Item Code	Format	Pack Size	Qualification
One Step H. Pylori Feces Test	W028-C	Cassette	25T/kit	CE
	W028P0026	Cassette	40T/kit	CE
	W028-S	Strip	50T/kit	CE
	W028P0027	Strip	100T/kit	CE